HEINE mini 3000® Diagnostic Sets D-851, D-852, D-856, D-873

CE 93/42/EWG/CEE/EEC

HEINE

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49(0)81 52/38-0 · Fax +49(0)81 52/38-202
E-mail: info@heine.com · www.heine.com
med 4706 1/10.09

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Warranty

Instead of the usual 2-year guarantee, we guarantee this device for 5 years from the date of delivery from our factory (this excludes consumables such as bulbs, tips and batteries). We guarantee the proper function of the unit provided it is used as intended by the manufacturer and according to the instructions for use. Any faults or defects which occur during the guarantee period will be repaired free of charge provided they are caused by faults in material, design or workmanship. In the case of a complaint of a defect in the product during the guarantee period, the buyer must prove that any such defect was present at the time the product was delivered. The usual legal guarantee and warranty do not apply to incorrect use, the use of non-original HEINE parts or accessories. (Especially bulbs, as these are designed with emphasis on the following criteria: Color temperature, life expectancy, safety, optical quality and performance.) Also excluded are repairs or modifications made by persons not authorised by HEINE or cases where the customer does not follow the instructions supplied with the product. Further claims, in particular claims for damage not directly related to the HEINE product, are excluded.

For **repairs**, please contact your supplier, who will return the goods to us or our authorized agent.

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HEINE XHL® Xenon Halogenlampe 2,5 V X-001.88.057

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ISO 15004-1 (Ophthalmic instruments - Fundamental requirements and test methods)

Instructions for use

ENGLISH

Read these instructions carefully before using the HEINE mini 3000® and keep them in a safe place for future reference.

In accordance with local regulations this product should be disposed of as an electronic device separately.

1. HEINE mini 3000® battery handle

Range of applications: The HEINE mini 3000® battery handle should only be used for operation to power HEINE mini 3000® instruments.

Batteries:

- 2 alkaline cells (AA/LR 6)
Important: Do not use alkaline manganese batteries with lithium as bulb life will be reduced.
- The HEINE mini 3000® battery handle can be charged in the HEINE mini NT® with the HEINE mini 2Z rechargeable battery 2,5 V.

Charging time:

- The mini NT indicates when the rechargeable battery is fully-charged. The illuminated ring is lit continuously.
 - mini 2Z rechargeable battery (2,5 V, NiMH) = ca. 4 h (730 mAh)

Instrument connector: Internal thread in handle head (1).

Switching on: Push switch (2) down – a red „on“ indicator comes into view.

Switching off: Push switch up. If the handle is put back firmly into a coat pocket the handle is automatically switched off.

Changing the batteries: Unscrew the end-cap (3) and shake out the old batteries/rechargeable battery. Insert new batteries/ rechargeable battery with the plus contact facing the handle head.

Cleaning: The handle can be cleaned with a damp cloth (e.g. alkaline or pH-neutral detergent). For cleaning disinfection of the outer surface we recommend the so called wipe disinfection by using detergents, which are released for Medical Devices made out of plastic. Disinfection by spraying or immersion as well as sterilisation is not allowed.

Important note:

We can only guarantee the performance and safety of HEINE mini 3000® instruments when they are fitted with original HEINE bulbs.

2. HEINE mini 3000® Ophthalmoscope

Range of applications: The HEINE mini 3000® ophthalmoscope is designed for examination of the eye. During examination, hold the instrument with your index finger on the lens wheel (4). You can also operate the aperture wheel (6) in this way. Window (5) shows the selected lens value. Minus lenses are shown red, plus lenses black.

Changing the bulb: Unscrew the instrument from the handle and pull the bulb (7) downwards. Wipe the new bulb glass clean and insert it so that the locating pin (8) fits the slit.

Cleaning: The housing can be wiped clean with a cloth moistened with alcohol. Glass surfaces can be cleaned with a cotton wool bud in the same way.

Spare Parts

HEINE XHL® Xenon halogen bulb X-001.88.106

3. mini 3000 Otokop/mini 3000 F.O. Otokop

Range of applications: The HEINE mini 3000® otoscope should only be used to examine the external ear, the nose and for general non-invasive examination. **Safety in use:** The otoscope should only be used to examine the ear or nose when a tip is fitted.

Use

Attaching a tip: The otoscope may only be inserted when a disposable (grey) or reusable (black) tip is fitted.

Push the tip onto the instrument so that the internal projection engages in the slit (9). Twist to the right to lock.

Magnifying lens: The lens (10) magnifies ca. 3x and can be swivelled aside for instrumentation.

Insufflation port: mini 3000 F.O. Otokop: An insufflation bulb can be fitted to the port.

General illumination: The otoscope can be used with or without a tip for general illumination of e.g. the oral cavity or skin. Avoid contact with sensitive or damaged skin.

Changing the bulb:

HEINE mini 3000® otoscope: Swivel the lens (10) to one side and simply pull the bulb backwards out of the head.

With the **HEINE mini 3000® F.O. otoscope**, first unscrew the head from the handle and pull the bulb (10a) downwards. Push in the new bulb.

Cleaning and Sterilization:

Otokop head: Clean the outside with a soft cloth, inside with a cotton wool bud, if necessary soaked in alcohol. Gas sterilization is permitted. Do not soak.

Reusable SANALON S tips: These can be cleaned, disinfected and sterilized or boiled by any conventional method. Autoclaving is permitted.

AllSpec-Tips: Are for single use only. Do not attempt to clean them as the tip may be damaged and could injure the patient.

Spare parts and accessories

HEINE XHL® Xenon halogen bulb mini 3000 X-001.88.110

HEINE XHL® Xenon halogen bulb mini 3000 F.O. X-001.88.105

4. HEINE mini 3000® laryngeal mirror

Safety informations

Range of applications: The straight laryngeal mirror is used to illuminate the mouth and throat during medical examination.

Safety in use: The curved light stem should never be inserted into the mouth without an attachment fitted to the end as the bulb can become hot. Switch off the instrument after use to avoid it becoming too hot.

Using the instrument: To avoid fogging of the mirror during use, the warmth from the bulb is usually sufficient. We recommend that the instrument be switched on a few seconds before the examination is made.

Changing the bulb: Unscrew first the mirror and then the bulb from the light stem.

Cleaning and sterilizing:

Do not place the light stem in liquid.

- 1. Cleaning:** Unscrew and remove the mirror from the light stem and clean both parts with a damp cloth – we recommend a soapy solution or alcohol.
- 2. Disinfection:** After cleaning, wipe all parts with a conventional disinfectant. Instrument heads can be soaked in disinfectant solutions, but should always be dried before use.
- 3. Sterilization:** Gas sterilization with ethylene oxide is recommended for all components. Autoclaving **should not be used for the mirror and light stem.**

Spare Parts and Accessories

HEINE XHL® Xenon Halogen spare bulb 2,5 V X-001.88.057

Applicable standards:

ISO 15004-1 (Ophthalmic instruments - Fundamental requirements and test methods)

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