# beurer

## GS XXL

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## Dear Customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for the applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Best regards,  
Your Beurer Team

## 1. Interesting facts

### Weighing correctly

If possible, always weigh yourself at the same time of day (ideally in the morning), after going to the toilet, on an empty stomach and without clothes to achieve comparable results.

## ! 2. Important notes – keep for later use!

### Caution – For your safety

- Attention! Do not step onto the scale with wet feet or when the scale's surface is damp—danger of slipping!
- You should protect the scale against impacts, moisture, dust, chemicals, major temperature fluctuations and keep it away from heat sources (ovens, radiators).
- Repairs may only be performed by Beurer Customer Service or authorized dealers. However, always check the batteries and replace them if necessary prior to making a complaint.
- All scales comply with EC Directive 89/336 + supplements. Should you still have questions regarding the use of our products, please contact your dealer or Beurer Customer Service.
- The scale should occasionally be cleaned with a damp cloth. Do not use aggressive cleaning agents and never submerge the scale.
- The device is only for private use and is not intended for medical or commercial purposes.
- On delivery the scale is set to the units “cm” and “kg”. A switch located on the back of the scale enables you to change to “inch”, “pound” and “stones” (lb, St).
- Please dispose of the appliance in accordance with the Directive 2002/96/EC on Waste Electrical and Electronic Equipment (WEEE). In case of queries, please contact the municipal authorities responsible for waste disposal in your area.


## 3. Use

This scale is used to determine the weight of human beings. The scale has a weighing range up to 440 lbs (200 kg), and can therefore also weight heavy persons. In addition, a target weight can be specified for 10 persons and the difference between the target weight and the actual weight is shown simultaneously. In this way you always have your desired weight before you. The extra-large glass step surface creates the condition for comfortable standing, even for heavy persons.

### 3.1 Batteries


If present, pull the battery insulating strip off the battery compartment cover or remove the battery's protective film and insert the battery according to the polarity. If the scale fails to operate, remove the battery completely and insert it again. Your scale is equipped with a “replace battery” indicator. When the scale is operated with a battery which is too weak, “Lo” appears in the display field and the scale automatically switches off. The battery must be replaced in this case (1 x 3 V lithium battery CR2032). The used, completely drained standard and rechargeable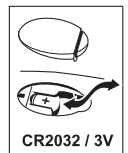

batteries must be disposed of in specially marked collection containers, at toxic waste collection points or electrical product retailers. You are legally obligated to dispose of the batteries.

Note: Batteries containing pollutants are marked with the following symbols: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

Place the scale on a firm, flat surface; a firm floor covering is a basic requirement for correct weight measurement.

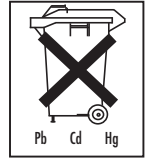### 3.2 Measuring weight only

To ONLY measure your weight without specifying a target weight, you must proceed as follows:

Quickly and firmly tap the standing surface of your scale with your foot. The entire display (Fig. 1) appears up to "0.0" (Fig. 2) as a self-test (tap-on technology).

Now the scale is ready to measure your weight. Stand on the scale without moving about and distribute your weight evenly on both legs. The scale immediately begins to measure your weight. The display "000" flickers until your weight has been measured (Fig. 3). The scale switches off 15 seconds after you step down from it. The measuring result remains visible during this time.

Fig. 1

Fig. 2

Fig. 3

### 3.3 Measuring weight with specified target

First you must store your personal target weight:

- Briefly step onto the scale first and wait until the display "0.0" (Fig. 2) appears. Then press "SET". Now the first memory preset flashes in the display (Fig. 4). Select the desired preset by pressing the buttons "up" ▲ or "down" ▼ and confirm the desired preset with the "SET" button. "0.0" appears in the display (Fig. 2).
- Now step onto the scale to measure your weight. With this weight measurement, a target weight is now automatically specified, which is 10 % below the determined value (Fig. 5). You can still change this value with the buttons "up" ▲ or "down" ▼.
- Confirm your entry with "SET". Your target weight and the memory preset are now briefly displayed again. The scale then automatically switches off.

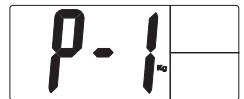
Fig. 4

To measure your weight taking your target weight into account, you must proceed as follows:

- Briefly step onto the scale first and wait until the display "0.0" (Fig. 2) appears.
- Select the memory preset where your basic personal target weight is stored by pressing the "User" button repeatedly. If no target weight is stored in a memory preset, "Err" appears at the upper right (Fig.6). The selected memory preset and your target weight are displayed until the display "0.0" (Fig. 2) appears.
- Now the scale is ready to measure your weight. Now step onto the scale. Stand still on the scale with your weight distributed evenly between both legs. The scale immediately begins to measure your weight. The display flickers until your weight has been measured. Shortly after this, the measuring result is shown in the large display field, the target weight in the upper right-hand field and the difference ( $\Delta$ =Delta) between the target weight and the measured weight in the lower right-hand field (Fig. 7). The scale switches off 15 seconds after you step down from it. The measuring result remains visible during this time.

Fig. 5

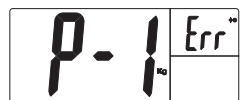
Fig. 6

Fig. 7

If the scale detects an error during weighing, "Err" appears in the display (Fig. 8). Make sure that you always switch on the scale first and wait for the display "0.0" (Fig. 2) before stepping on.

Fig. 8

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